

Face Shield Cleaning Instructions

- After doffing, hand hygiene, remove dirty gloves, hand hygiene, put on new gloves.
- Prepare to clean the shield with the purple top Super Sani-Cloth.
- Wipe the inside followed by the outside.
- **Ensure the surfaces are wet for a contact time of two (2) minutes.**
- If the cleaning cloth leaves a residue, the shield can be wiped with clean water or alcohol to remove the residue **AFTER** the appropriate contact time has been achieved.
- Fully dry: air dry or a clean absorbent towel. Remove gloves and hand hygiene.
- Place decontaminated shield in a designated clean space.

1. Hold the bottom of the halo with your thumb



2. Hold the top of the halo with your remaining fingers as shown



3. A proper grasp of the halo shield is shown above



4. Apply light pressure to clean the shield



NOTE: Shield material is not meant for high impact or load bearing. **DO NOT** store under weight in lockers, duffle bags, etc...



NOTE: Avoid applying pressure to the sides of the folded area of the shield to prevent cracking

